FR

EMBOUT SIFFLO +

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EN
MOUTHPIECE SIFFLO +

PLEASE READ CAREFULLY BEFORE USE AND KEEP FOR FUTURE REFERENCE.

Safety precautions : The SIFFLO+ mouthpiece is an accessory of the LORIO medical device. Before using the LORIO medical device, please refer to the user's manual included in the STARTER SIFFLO / SIFFLO+ pack or available on the our website www.happlyzmedical.com


Don't use the SIFFLO+ mouthpiece for anything other than its intended purpose.

Don't lend your SIFFLO+ mouthpiece ! It is strictly personal. However, for collective use, it is possible to fit a single-use 22F/22M intersection antimicrobial filter (not supplied).

Be sure to clean the mouthpiece after each use to avoid the risk of respiratory infections.

Indications : The SIFFLO+ mouthpiece is a round mouthpiece designed to reproduce open-glottis breathing. This mode of ventilation is particularly recommended for bronchial drainage of congested patients.


Assembly of the mouthpiece : The mouthpiece (**1.a** + **1.b**) can be pushed in and out of place by inserting or pulling it over the studs on the top of the flute. Be careful not to apply too much pressure. Pre-assemble the mouthpiece (combination of **1.a** with **1.b**) before inserting it on the flute.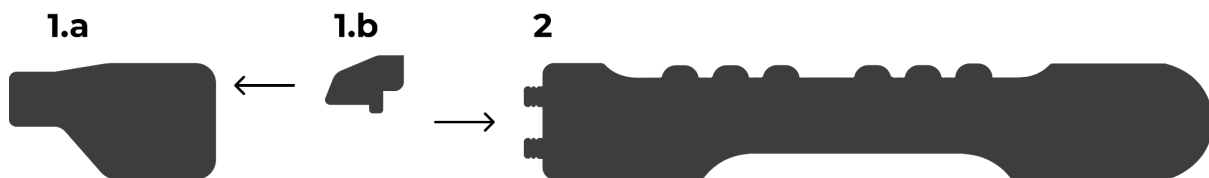

Cleaning and care : SIFFLO+ is a removable mouthpiece specially designed for optimum sterilization of its inner and outer surfaces, enabling the LORIO device to be used in complete safety.

Before the very first use and after each use, disinfect the mouthpiece (**1.a** + **1.b**).


For optimum disinfection, remove the inner part (**1.b**) from the outer part (**1.a**) as shown in the image below.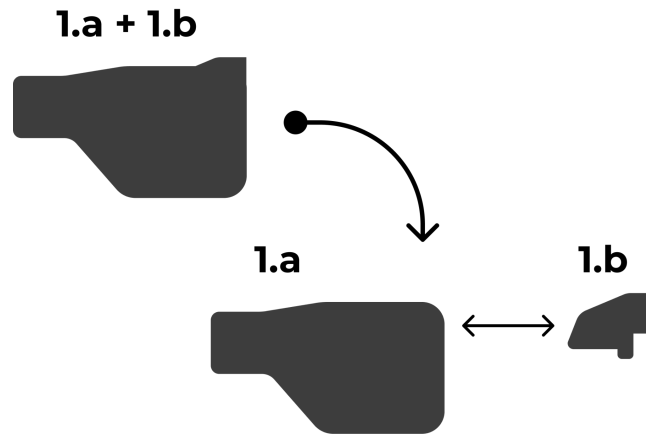

For individual, personal use of the mouthpiece : Wash the mouthpiece in warm, soapy water and let it air-dry on a clean, disinfected surface. Do not clean in the dishwasher.

For intensive cleaning : In a suitable container, prepare a bath of Surfanios Premium (or similar) as follows:

1. Dilute a 20ml bag of Surfanios Premium in a sufficiently large tub and top up with 4 liters of sterile water,
2. Immerse both parts of the mouthpiece in the bath (several mouthpieces can be sterilized at the same time),
3. Leave for 30 minutes,
4. Rinse with clean water,
5. Remove both parts of the mouthpiece and leave to dry on a clean, disinfected surface.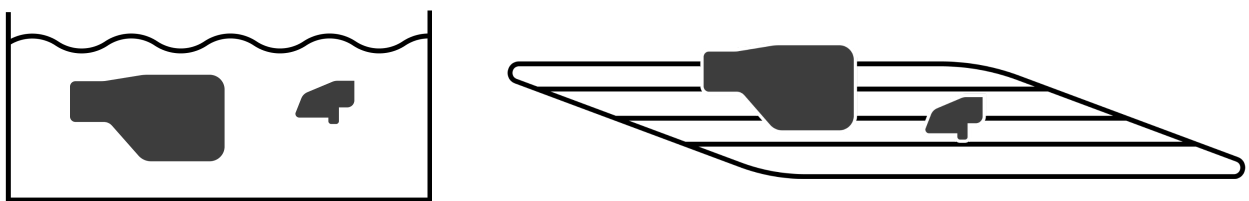

Once disinfected and both parts of the mouthpiece (**1.a + 1.b**) dry, assemble your mouthpiece (**1.a + 1.b**) and combine it with your flute (**2**) in the direction shown in the diagram above. Your device is now assembled and ready to use !

Recycling : The SIFFLO+ mouthpiece is made of biocompatible polycarbonate for medical use, and is a recyclable plastic.

At the end of its life, dispose of it in the appropriate recycling bin.